



CleanBath for QuickThaw® Plasma Thawing Systems

For your convenience, Helmer offers CleanBath, a product that will help keep your water bath clean and odor-free. While intended for use in the Helmer Plasma Thawing Systems, CleanBath may also be used in other laboratory water baths. Each 8-oz bottle of CleanBath treats approximately 300 gallons of water and is compatible with stainless steel baths. Contact Helmer (USA and Canada) or your local distributor to order CleanBath today.

Order Your 8 oz. Bottle by Contacting:

USA & Canada:

Phone: +1.800.743.5637
Fax: +1.317.773.9082
Website: www.helmerinc.com
E-mail: sales@helmerinc.com

All other countries

Please contact your local distributor

Instructions for use

Use 12 drops of CleanBath per gallon of water.

Helmer Plasma Thawing System complete water change requirements:

| | | |
|------|------------|----------------|
| DH2 | (2.2 gal) | requires 0.8cc |
| DH4 | (4.75 gal) | requires 1.7cc |
| DH8 | (8.5 gal) | requires 3.0cc |
| DH24 | (14 gal) | requires 5.0cc |